How to use your new Rehband Support - 7910

UK

Function

Light, warming support for the wrist. Allows a free grip for the hand.

Indication

Easy overstrains and preventing injury.

Material

1,5 mm neoprene/SBR, polyamidejersey.

Application

- 1) Put the thumb through the opening.
- 2) Pull the string behind the back of the hand and wind around the wrist.
- 3) Close the Velcro.

Important Information

- 1) If you have questions regarding application, please contact your local dealer or the prescriber.
- 2) The product is not flameproof, avoid temperature above 120°C/ 248°F.
- 3) The function of the product must not be changed or compromised, the product should only be used for its original, intended indication.
- 4) Re: neoprene products, time of use should not exceed 3-4 hours. Re: products in other materials, time of use should not exceed 8 hours, unless otherwise is recommended by prescriber.
- 5) If the product contains latex or rubber and you are allergic to latex/ rubber, we advise you to consult a prescriber before using the support. The content of the product is written above.
- 6) If you are allergic to heat, we advise you to use a product that not contains neoprene.

Washing Instructions

- 1) Close all Velcro.
- 2) Mesh laundry bag is recommended.
- 3) Stretch product while wet and let dry.

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